



# N-95 MASKS

N-95 masks should be prioritized for high risk patient contact

**1** WASH HANDS BEFORE AND AFTER REMOVAL OF MASK

**2** PHYSICALLY INSPECT MASK BEFORE USE

**3** ENSURE PROPER FIT - NEEDS TO COVER NOSE AND MOUTH

**4** PLACE LOWER STRAP BEHIND NECK

**5** PLACE TOP STRAP ON BACK OF HEAD

**6** AVOID TOUCHING INSIDE OF MASK

**7** IF MASK CONTAINS METAL NOSE BAND, BEND METAL TO SHAPE AROUND NOSE WITHOUT PINCHING AIRWAY

**8** STORE MASK IN DESIGNATED AREA (BROWN PAPER BAG) OR UNDER CONTROLLED CONDITIONS (UV LIGHT)

**9** ENCOURAGE PRACTICAL HYGIENE REUSE OF N-95 MASKS - SHOULD NOT EXCEED 5 USES

**10** DISCARD APPROPRIATELY WHEN USE HAS EXPIRED

REFERENCES: PLEASE VISIT  
[HTTPS://MANAGEDCAREPHARMACYCA.COM](https://MANAGEDCAREPHARMACYCA.COM)



# SURGICAL MASKS

**1** WASH HANDS BEFORE AND AFTER REMOVAL OF MASK

**2** PHYSICALLY INSPECT MASK BEFORE USE

**3** ENSURE PROPER FIT - STARTS AT NOSE BRIDGE AND ENDS DOWN PAST THE MOUTH

**4** AVOID TOUCHING INSIDE OF MASK

**5** FOLD ENTRANCES SHOULD BE FACING DOWNWARD

**6** PLACE LEFT ELASTIC BEHIND LEFT EAR

**7** PLACE RIGHT ELASTIC BEHIND RIGHT EAR

**8** STORE MASK IN DESIGNATED AREA (BROWN PAPER BAG) OR UNDER CONTROLLED CONDITIONS (UV LIGHT)

**9** ENCOURAGE PRACTICAL HYGIENE REUSE MASK

**10** DISCARD APPROPRIATELY WHEN USE HAS EXPIRED (24 HOURS)

EMAIL: [MHATA@WESTERNU.EDU](mailto:MHATA@WESTERNU.EDU) OR  
[KEVIN.MANCILLA@WESTERNU.EDU](mailto:KEVIN.MANCILLA@WESTERNU.EDU)